

Product data sheet





Information

Designation ARSIS T- or L-shaped shower bar, Anthracite grey Epoxy-coated


Aluminium

Reference 049721

Product advantages

- + Adjustable: convertible into a T or L shape, for all configurations.
- + Elliptic tube for an improved grip.
- + Help to get up and stand.
- + With slider bracket shower handset.

Description

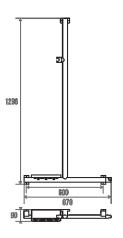

Elliptical tube in anthracite grey epoxy-coated aluminium, 38 x 25 mm.

Anthracite grey epoxy-coated resin flanges and cover plates.

Adjustable: convertible inot a T- or L-shape at assembly stage. Once mounted, the positionning remains fixed.

Concealed fixings.

Screws included.

Anticorrosion warranty.

Design coordinated with all ARSIS Comfort items.

Perfectly suitable for use in the hospitality industry.

Technical characteristics

Finish
Color
Dimensions
Diameter Ø

Gross weight Packaging

Material

CE Marking : safe load Laboratory tests Epoxy-coated
Anthracite grey
1296 x 637 mm
38 x 25 mm

Aluminum 2.020 kg

Cardboard box + label

150 kg 225 ka





 Made in France
 yes

 Maintenance
 yes

 Guarantee
 10 years

 Net weight
 1.500 kg

 Gencod
 3563390497210

 Custom code
 7615200000

Technical sheet <u>A04982007_IND_10</u>

Spare parts Réf. 049007 : Support de fixation alu chromé

Fixing and maintenance advices

Recommended height for the horizontal part: 800 mm from the ground. Grab bars provide support and added safety if properly fitted in accordance with our technical instructions. Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. DO NOT use the grab bar as a step. DO NOT sit on or hang from the grab bar.

Fiche produit mise à jour le : 10/01/2024