

Product data sheet





Information

Designation ARSIS Two-wall corner grab bar, White Epoxy-coated Aluminium,


mat chrome-plated flanges

Reference 049930

Product advantages

- + Make stepping into or out of the shower easier.
- + Elliptic tube for an improved grip.
- + Possible to fit it "to the right" or "to the left".
- + Concealed fixings.

Description

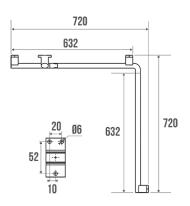

Elliptical tube in white epoxy-coated aluminium, 38 x 25 mm.

90° angled.

Mat chrome-plated resin cover plates.

3 fixing points. 3rd point to be fitted for right or left version.

Concealed fixing.

Screws included.

Anticorrosion warranty.

Design coordinated with all ARSIS bathroom accessories.

Perfectly suitable for use in the hospitality industry.

Also available with mat white epoxy-coated coverplates, ref. 049830.

Technical characteristics

Finish

Color

Dimensions
Diameter Ø

Material

Gross weight

Packaging

CE Marking : safe load Laboratory tests Epoxy-coated

Mat chrome-plated & white

632 x 632 mm

38 x 25 mm


Aluminum

1.730 kg

Cardboard box

150 kg

225 kg

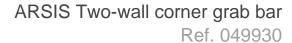



Made in FranceyesMaintenanceyes

 Guarantee
 10 years

 Gencod
 3563390499306

Technical sheet A04983001 IND 03

Spare parts Réf. 049007 : Support de fixation alu chromé, Réf. 049009 : Cache-fixation gris chromé mat

7615200000

Fixing and maintenance advices

Recommended height for the horizontal part: 800 mm from the ground. Grab bars provide support and added safety if properly fitted in accordance with our technical instructions. Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. DO NOT use the grab bar as a step. DO NOT sit on or hang from the grab bar.

Fiche produit mise à jour le : 10/01/2024

Custom code