

## Product data sheet




### Information

Designation	ERGONOMIC shower seat with adjustable height
Reference	047836

### Product advantages

- + **Adjustable legs:** it adjusts to the gradient of the shower base.
- + **Comfortable:** large seat with curved slats and smooth cushion.
- + **Maximum comfort:** with backrest and armrests.
- + **Ideal for each place** space saving when folded.
- + **Optimum hygiene:** removable smooth cushion.

### Description

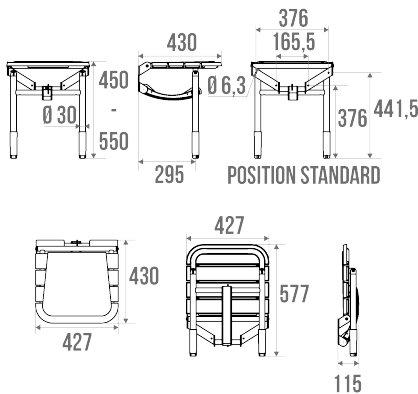

Foldaway seats with legs whose height can be adjusted to 100 mm and that are equipped with smooth shoes for 450, 500 to 550 mm height (it adjusts to the gradient of the shower base).

White epoxy-coated aluminium tube, Ø 25 mm.

Large size ergonomic seat, 4 curved slats in white polypropylene. Cushions in grey polyurethane like RAL 7012.

Enables users to sit down safely and comfortably while showering.

### Technical characteristics

Finish	Epoxy-coated
Color	White
Dimensions	427 x 450 x 430 mm
Diameter Ø	25 mm
Material	Aluminum
Gross weight	3.600 kg
Packaging	Cardboard box + Label
CE Marking : safe load	150 kg

Laboratory tests	225 kg
Made in France	yes
Maintenance	yes
Guarantee	10 years
Net weight	2.790
Gencod	3563390478363
Custom code	7615200000
Technical sheet	<a href="#">S00002132_IND_03</a>

## Fixing and maintenance advices

---

Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. The shower seat provides support and added safety if properly fitted in accordance with our technical instructions. For use in seated position. To guarantee the user safety, DO NOT rise up on the seat. When the seat is raised, DO NOT use it as a grab bar. To avoid any risks of pinching out, DO NOT put your fingers in the mobile parts. The product does not contains any dangerous substance for the health. We disclaim all liability for any personal injury resulting from inappropriate installation or use of this seat.

Fiche produit mise à jour le : 10/01/2024