

Product data sheet






Designation Reference

Taupe and chrome grey foldaway shower seat 047629

Product advantages

- + Maximum safety: with automatic stand.
- + Compact in raised position.
- + Easy fixing
- + Corrosion resistant warranty.

Description

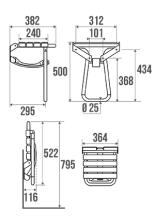

Standard size. 4 taupe (Pantone 7591U) polypropylene slats. Grey epoxy-coated aluminium tube. Enables users to sit down safely and comfortably while showering.

Standard height 500 mm.

Technical characteristics

Finish	Epoxy-coated
Color	Chrome Grey & Taupe
Dimensions	380 x 355 x 500 mm
Diameter Ø	25 mm
Material	Aluminium & Polypropylene
Gross weight	2.600 kg
Packaging	Cardboard box + label
CE Marking : safe load	150 kg
Laboratory tests	225 kg

PELLET ASC - ZI DE CHAPOTIN - 240 RUE DES FRÈRES VOISIN - CS 270048 - 69967 CHAPONNAY CEDEX - FRANCE - TEL : +33 (0)4 78 96 82 20 - FAX : +33 (0)4 78 96 70 52

© PELLET ASC – Non contractual document. Please read our General Sales Conditions. Dimensions in mm.

All dimensions and specifications are given for information purposes only and can be changed without notice in order to improve our products.



Made in France Maintenance Guarantee Net weight Gencod Custom code Technical sheet Spare parts yes yes 10 years 2.174 kg 3563390476291 761520000 A13470022_IND_21 Ref. 047628, Taupe (Pantone 7591U) seat slat. Ref. 047644,Foot cap for shower seat. Ref. 047623, Support slat.

Fixing and maintenance advices

Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. The shower seat provides support and added safety if properly fitted in accordance with our technical instructions. For use in seated position. To guarantee the user safety, DO NOT rise up on the seat. When the seat is raised, DO NOT use it as a grab bar. To avoid any risks of pinching out, DO NOT put your fingers in the mobile parts. The product does not contains any dangerous substance for the health. We disclaim all liability for any personal injury resulting from inappropriate installation or use of this seat.

Fiche produit mise à jour le : 10/01/2024