

# Product data sheet





#### Information


Designation Reference

Straight grab bar, 800 mm, White & Red Epoxy-coated Aluminium , tube Ø 30 mm 049182

#### **Product advantages**

- + Invisible fasteners.
- + Anti-corrosion guarantee.

#### Description

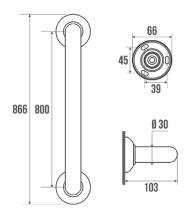

Concealed fixings. White epoxy-coated aluminium tube Ø 30 mm. Red (RAL 3003) resin flanges Ø 65 mm.

Possibility to fix the toilet roll holder ref. 048820 to this bar. Design coordinated with the TRIOLO Range accessories. Fixing screws included (4 x 30 mm).

### **Technical characteristics**

Finish	Epoxy-coated
Color	White & Red
Dimensions	800 mm
Diameter Ø	30 mm
Material	Aluminum
Gross weight	0.620 kg
Packaging	Plastic protection bag + Hanger for display
CE Marking : safe load	120 kg
Laboratory tests	200 kg
Made in France	yes
Maintenance	yes
Guarantee	10 years

PELLET ASC - ZI DE CHAPOTIN - 240 RUE DES FRÈRES VOISIN - CS 270048 - 69967 CHAPONNAY CEDEX - FRANCE - TEL : +33 (0)4 78 96 82 20 - FAX : +33 (0)4 78 96 70 52

© PELLET ASC – Non contractual document. Please read our General Sales Conditions. Dimensions in mm. All dimensions and specifications are given for information purposes only and can be changed without notice in order to improve our products.



Gencod Custom code Technical sheet 3563390491829 7615200000 A49130137\_IND\_07

## Fixing and maintenance advices

Recommended height for the horizontal part : 800 mm from the ground. Grab bars provide support and added safety if properly fitted in accordance with our technical instructions. Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. DO NOT use the grab bar as a step. DO NOT sit on or hang from the grab bar.

Fiche produit mise à jour le : 09/03/2023