

Product data sheet






Information

Designation Reference T-shaped shower bar, 600 x 1000 mm, White & Red Epoxy-coated Aluminium , tube Ø 30 mm 049342

Product advantages

- Support and lifting aid.
- With sliding shower holder.
- Invisible fastenings.
- Anti-corrosion warranty.

Description

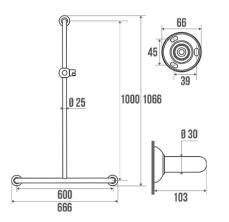

Supplied with universal clip-on shower rail slider bracket in white ABS. White epoxy-coated aluminium tube. Red (RAL 3003) resin cover plates.

Design coordinated with the TRIOLO range accessories and with the COMFORT and SAFETY items.

Technical characteristics

Finish	Epoxy-coated
Color	White & Red
Dimensions	600 x 1000 mm
Diameter Ø	30 mm
Material	Aluminum
Gross weight	1.340 kg
Packaging	Plastic protection bag + Hanger for display
Maximum Weight Capacity	150 kg
CE Marking : safe load	150 kg
Laboratory tests	200 kg
Made in France	yes

PELLET ASC - ZI DE CHAPOTIN - 240 RUE DES FRÈRES VOISIN - CS 270048 - 69967 CHAPONNAY CEDEX - FRANCE - TEL : +33 (0)4 78 96 82 20 - FAX : +33 (0)4 78 96 70 52

All dimensions and specifications are given for information purposes only and can be changed without notice in order to improve our products.



Maintenance Guarantee Gencod Custom code Technical sheet yes 10 years 3563390493427 7615200000 A49130137 IND 07

Fixing and maintenance advices

Recommended height for the horizontal part : 800 mm from the ground. Grab bars provide support and added safety if properly fitted in accordance with our technical instructions. Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. DO NOT use the grab bar as a step. DO NOT sit on or hang from the grab bar.

Fiche produit mise à jour le : 10/01/2023