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**Do not install this equipment before reading the following instructions:** **Fitting instructions.** Before drilling the wall, physically position the product (in the lowered position) against the wall so that the stand is PERPENDICULAR to the floor; then mark the positions of the holes to be drilled. Pegs have to be selected according to the type of wall. The fixation points have to be imperatively respected. **Cleaning and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and user's safety, we recommend the use of mild soaps when cleaning. **CAUTION: DO NOT USE** cleaning products made of: ketones, hydrocarbons, acids, bases, esters and ethers. After the use of a cleaning product, rinse with clear water. **Safety warnings.** The shower seat provides more security to the user if it is fitted according to instructions. After fitting and before using, make sure that fixing screws are tightened securely. The shower seat is designed to be used in a seated position. To guarantee the user's safety, it is FORMALLY FORBIDDEN to climb and stand on the seat. When the seat is in a raised position, DO NOT USE it as a grab bar. To avoid any pinching risks, do not put fingers in the mobile parts. It is formally FORBIDDEN to use the armrest as a grab bar or to make a transfer. **The product does not contain any hazardous substances to health. The manufacturer disclaims all responsibility in case of injury caused by improper installation or use of this shower seat.** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

Vous aurez besoin / You will need			

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