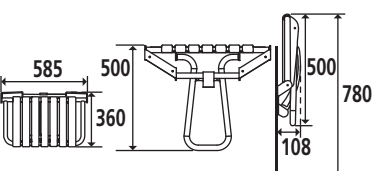
 Poids maximum utilisateur / Max. user weight / Maximumgewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht / Макс. вес пользователя.
 Poids du produit / Product's weight / Productgewicht / Peso del producto / Peso del prodotto / Peso do produto / Gewicht des Produktes / Вес изделия

Ce produit est conforme avec la directive 93/42/EEC des dispositifs médicaux.
 This product is conform with the directive 93/42/EEC for medical products.
 Dit product beantwoordt aan Richtlijn 93/42/EEC over medische apparatuur.
 Este producto es conforme con la Directiva 93/42/EEC relativa a los productos médicos.
 Questo prodotto è in conformità con la Direttiva 93/42/EEC relativa ai dispositivi medici.
 Este produto está conforme à Directiva 93/42/EEC relativa aos dispositivos médicos
 Dieses Erzeugnis entspricht der Richtlinie 93/42/EEC über Medizinprodukte.
 Соответствует требованиям директивы 93/42 ЕС о медицинских изделиях.

Référence P / Reference P / Referentie P / Referencia P / Codice P / Referència P / Referenz P / Код P :	047632 047633 047634 047643
--	--------------------------------------

Fonction	Permet de prendre une douche en position assise.
Function	Allows to take a shower in a seated position.
Functie	Zo kan u zittend een douche nemen.
Función	Permite tomar una ducha en posición sentada.
Funzone	Permette di prendere una doccia in posizione seduta.
Função	Permite tomar um ducho sentado.
Funktion	Ermöglicht es, im Sitzen zu duschen.
Назначение	Позволяет принимать душ в положении сидя.

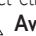
Descriptif		 150 kg
Description		 2,990 kg
Beschrijving		
Descripción		
Descrizione		
Descrição		
Beschreibung		
Описание		

Número notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	A13600811
--	-----------

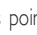
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	10/10/2002
--	------------

Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции	15
--	----

Date / Date / Datum / Fecha / Data / Data / Datum / Дата	23/10/2018
--	------------

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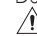
GB  **Do not install this equipment before reading the following instructions:**  **Fitting instructions.** Before drilling into the wall, physically hold the product (in the lowered position) against the wall so that the stand is PERPENDICULAR to the floor, then mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The 4 fixings points have to be necessarily respected.  **Care and maintenance.** IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** The shower seat adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. The shower seat is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. When the seat is in a raised position, DO NOT USE it as a grab bar. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this shower seat.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS