Référence P / Reference P / Referentie P / Referencia P / Codice P / Referència P / Referenz P / Код P :		048809 048810
Fonction	Permet à l'utilisateur de prendre appui en toute sécurité.	
Function	Allows a safety leaning.	
Functie	Zo kan de gebruiker in alle veiligheidsomstandigheden ondersteund worden.	
Función	Permite de apoyarse con seguridad.	
Funzione	Permette di prendere appoggio con sicurezza.	
Função	Permite ao utilizador apoiar-se em toda a segurança.	
Funktion	Zum sicheren Abstützen des Benutzers.	
Назначение	Служит для пользователя надежной опорой	
Descriptif		150 kg
Description		
Beschrijving		
Descripción		
Descrizione		
Descrição		
Beschreibung		
Описание		
Número notice / Instruction sheet no / Fichenummer / Número instrucciones / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции		S00001398
Date de création / Creation's date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания		06/03/2002
Révision n° / Revision no. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / No редакции		07
Date / Date / Datum / Fecha / Data / Data / Datum / Дата		02/07/2015

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Do not install this equipment before reading the following instructions:

Fitting instructions. Our support prop is suitable for all our hinged bars. The hinged

bar has to be fitted so that the upper part of the bar (in horizontal position) is to 80 cm of the ground. To guarantee the good use of this product, we recommend to fit it according to instructions.

Care and maintenance. IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.

Safety warnings. The support prop adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the stabilization screw is well engaged. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this support prop.

At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS